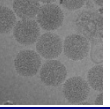
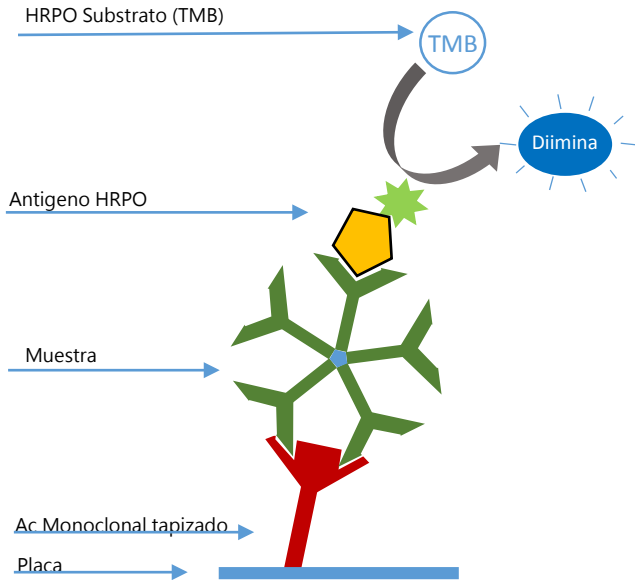
INgezim FVR IgM

R.13.FVR.K2

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

INgezim FVR IgM

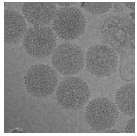R.13.FVR.K2

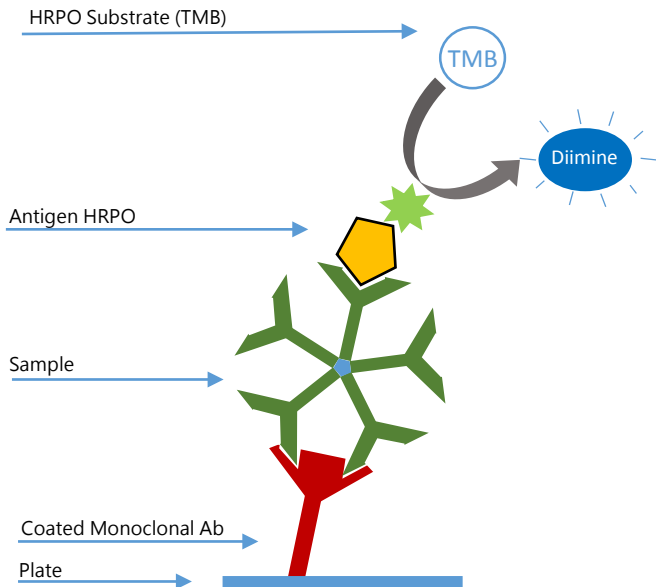INgezim FVR IgM is a capture immunoenzymatic assay which uses a monoclonal antibody specific to ruminant IgM and a Rift Valley Fever Virus (RVFV) recombinant N protein conjugated with HRPO.

TECHNICAL BASIS OF THE KIT

1. Plates are coated with a monoclonal antibody (MAb) specific to ruminant IgM. Samples are added and incubated.
2. If the sample contains IgM (specific or not), they will be captured by the α -ruminant IgM MAb coating the plate.
3. Once the RVFV conjugated antigen is added, the N protein will be captured by the specific IgMs contained in the sample.
4. This binding is detected by adding a specific substrate.

APPLICATION

Detection of specific IgM antibodies to RVFV N protein

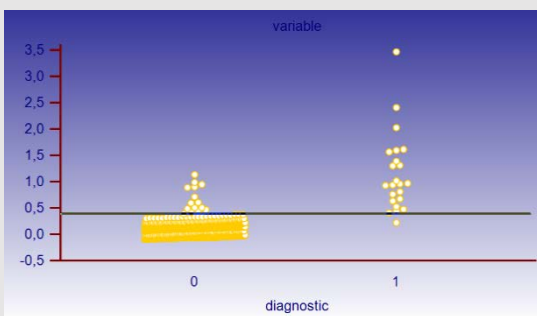
INTERPRETATION OF THE RESULTS

The assay establishes one cut off. Samples will be considered **Positive** (i.e. contains specific IgM antibodies to RVFV), if their positivity index is higher than the cut off. Samples will be considered **Negative** (i.e. do not contain specific IgM antibodies to RVFV), if their IP value is lower than the cut off.

VALIDATION


DIAGNOSTIC SPECIFICITY AND SENSITIVITY

1589 ovine, caprine and bovine sera were analyzed. Positive samples were taken from experimentally infected and vaccinated animals. The negative samples corresponded to different RVFV free areas in Spain. **The assay showed 95.7% sensitivity and 99.3% specificity.**

ANALYTICAL SENSITIVITY

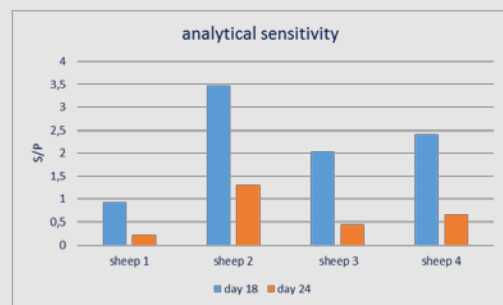2 extractions made at different days post infection (18 & 24 p.i.) to 4 sheep experimentally infected with RVFV, were analyzed. The analytical sensitivity to detect specific IgMs was at least day 18 p.i. (*Data of previous days were not available*)

COMPOSITION OF THE KIT

- Microtitration plates of 96 wells
- Vials with positive antigen HRPO conjugated
- Vials with Positive Control
- Vials with Negative Control
- Bottle with Washing Solution
- Bottle with diluent
- Bottle with stop solution
- Bottle with substrate (TMB) ready to use

PRODUCT MANUFACTURED BY INGENASA

SHELF LIFE: 12 months
Stored at 2°C-8°C

Ed.020217