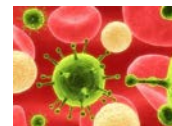
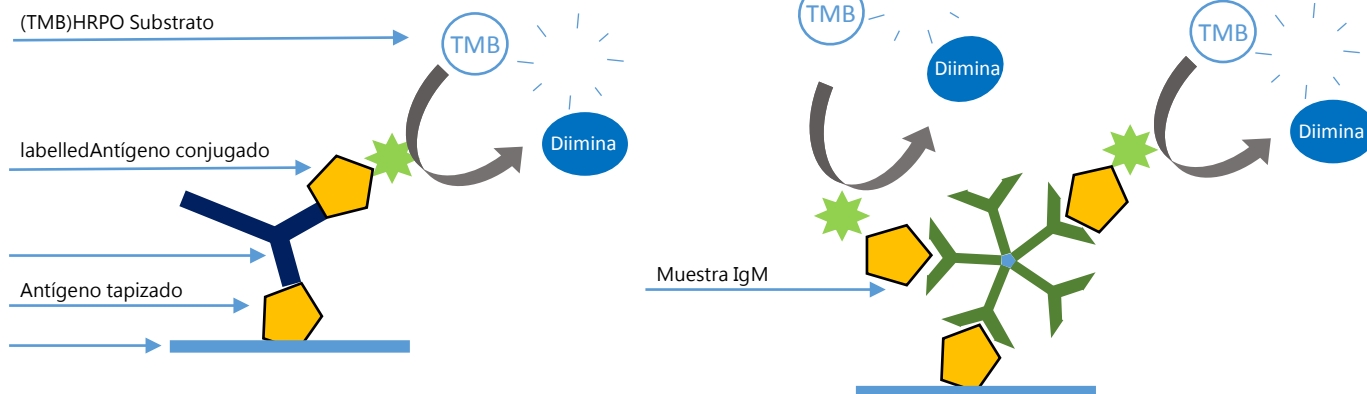
INgezim ANEMIA EQUINA

R.14.AIE.KO

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
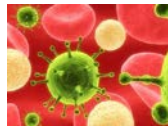
INgezim ANEMIA EQUINA

R.14.AIE.KO

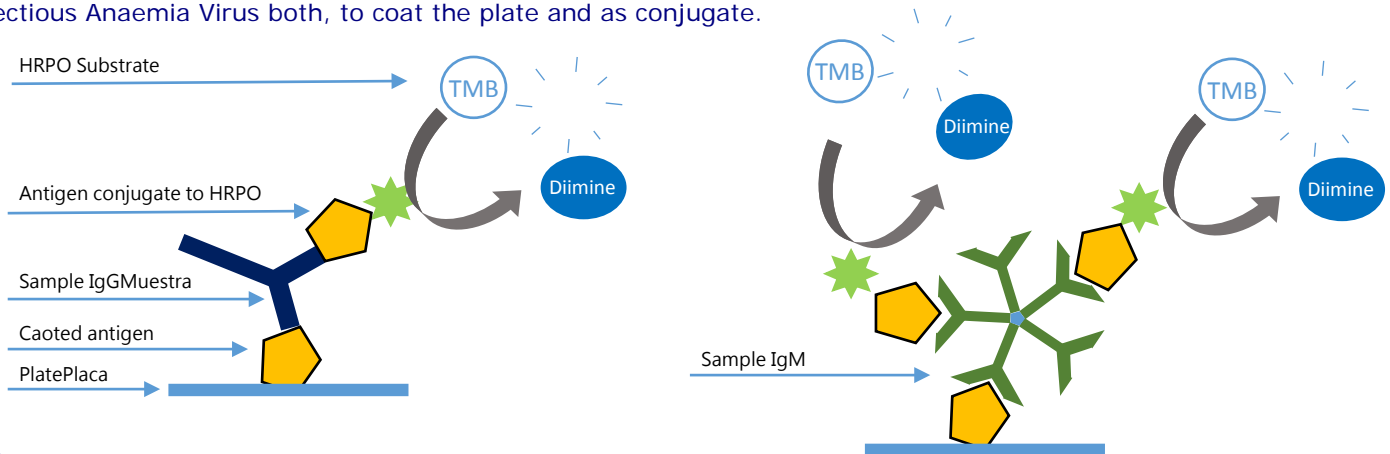INgezim ANEMIA EQUINA is an immunoenzymatic assay which uses the recombinant protein p26 of Equine Infectious Anaemia Virus both, to coat the plate and as conjugate.

TECHNICAL BASIS OF THE KIT

1. Plates are coated with p26 recombinant protein. Serum samples are added to the wells and incubated.
2. If the sample contains specific antibodies to EIA, they will bind to the antigen.
3. When a HRPO conjugated recombinant p26 protein is added, it will bind to the antibodies present in the sample if the serum is positive. This binding is detected by adding a specific substrate.

APPLICATION

Detection of specific antibodies to p26 protein of EIA Virus.

INTERPRETATION OF THE RESULTS

The assay establishes one cut off. Samples will be considered **Positive**, if their OD value is higher than the cut off. Samples will be considered **Negative**, if their OD value is lower than the cut off.

SENSITIVITY

Study 1. 14 sera samples of EIAV naturally infected horses were analyzed. Samples belonged to the last outbreak in Spain (1983) and from other outbreaks in Argentina. **The results obtained showed 100% sensitivity.**

Study 2. 55 AGID positive sera were analyzed (27 strong positive, 28 weak positive). **The sensitivity of the assay in relation to AGID was 96%.**

ANALYTICAL SENSITIVITY

Serial dilutions of 4 AGID strong positive sera were analyzed. The assay showed lower sensitivity than AGID in one serum (1 dilution 1/64), similar for another one (1/64) and higher than AGID for 2 sera (1 dilution 1/128 & 1/256).

COMPOSITION OF THE KIT

- Microtitration plates of 96 wells
- Vials with Positive Control
- Vials with Negative Control
- Vials with Peroxidase Conjugate
- Bottle with Washing Solution
- Bottle with diluent
- Bottle with stop solution
- Bottle with substrate (TMB) ready to use

PRODUCT MANUFACTURED BY INGENASA
REGISTRATION NUMBER 2340 RD

IT-73840 ISO 14001:2015 ISO 9001:2015
IT-73780 9191.INGE 9175.ING2

SHELF LIFE: **18 months**
Stored at 2°C-8°C

Ed.250117