
INgezim APP mix 10-12

R.11.APX.K1

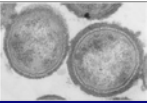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Referencias: Evaluation of diagnostic assays for the serological detection of *Actinobacillus pleuropneumoniae* on samples of known or unknown exposure. Opriessnig T. et al. (2012). J. of Vet. Diag. Invest. XX(X) 1-11

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

INgezim APP mix 10-12

R.11.APX.K1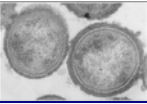

INgezim APP 10-12 mix is an immunoenzymatic assay based on a Indirect ELISA technique for the detection of antibodies against *Actinobacillus pleuropneumoniae* serotypes 10 and 12 in porcine serum.

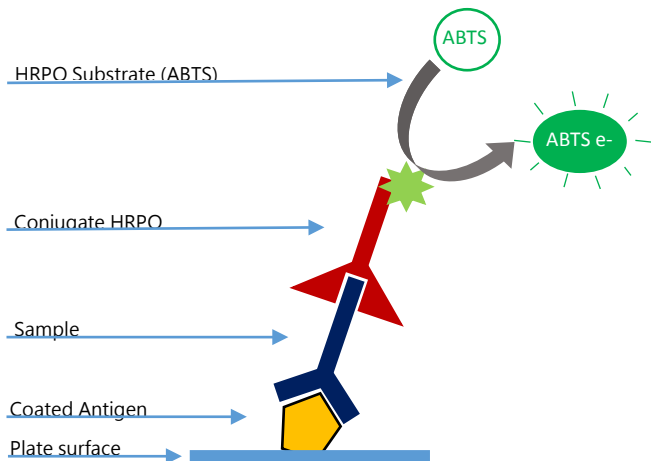TECHNICAL BASIS OF THE KIT

1. Plates are coated with APP antigen serotypes 10 and 12. The sample is added to the wells and incubated.
2. If the sample is positive and contains specific antibodies to APP, they will bind to the antigen.
3. Once the conjugate is added, it will bind to the porcine immunoglobulins present in the positive sample. The binding is detected by the development of a colorimetric reaction after the addition of the substrate.

APPLICATION

INgezim APP mix 10-12 has been designed to detect antibodies specific to APP 10 and 12 in individually assayed porcine sera samples.

INTERPRETATION OF THE RESULTS

The assay uses two Cut Offs. Samples showing OD values higher than or equal to the positive cut off will be considered **positive**. Samples showing OD values lower than the negative Cut off must be considered **negative**. Samples showing OD values in the range between both Cut off must be considered **doubtful**.

SENSITIVITY AND SPECIFICITY

339 sera were analyzed by INgezim APP mix 10-12 and the results obtained were compared with the ones obtained by specific ELISAs which had been previously evaluated against the Reference ELISA LC-LPS ELISA designed at Marcelo Gottschalk's Laboratory (College of Veterinary Medicine, Universidad de Montreal). The relative sensitivity (Se) and specificity (Sp) in relation to the specific assays are shown below:

	APP10		APP12	
	Doubtful (+)	Doubtful (-)	Doubtful (+)	Doubtful (-)
Se	100%	89,47%	100%	89,58%
Sp	72%	100%	72,6%	92,08%
N°	142		197	

References: Evaluation of diagnostic assays for the serological detection of *Actinobacillus pleuropneumoniae* on samples of known or unknown exposure. Opriessnig T. et al. (2012). J. of Vet. Diag. Invest. XX(X) 1-11

COMPOSITION OF THE KIT

- Microtitration plates of 96 wells
- Vials with Positive control
- Vials with Negative control
- Vials with Conjugate
- Bottle with washing solution
- Bottle with diluent
- Bottle with substrate
- Bottle with stop solution

PRODUCT REGISTERED IN SPAIN
(3043RD) DISTRIBUTED IN SPAIN BY
INGENASA

IT-73840 ISO 14001:2015 ISO 9001:2015
IT-73780IT- 9191.INGE9 9175.ING29

SHELF LIFE: **12 months**
Store at 2-8°C

Ed. 020217